

# LASCOD

# BIO

degradable

# ALL CLEANER THE BIO SOLUTION



# Affordable and easy to use, now with a fresh **mint fragrance!**



## neutral pH

## efficacy and respect for nature

The residues removal and the correct detercion are the basic steps for a correct use of the instruments.

Above all that, Lascod has a special consideration for the environmental conservation.

The formulation of All Cleaner is based on the new generation of active ingredients enviromentally friendly.

Cleaning agents contained in the powder are completely biodegradable.




## without phosphates and surface-active agents



**LAS**

# affordable and easy to use

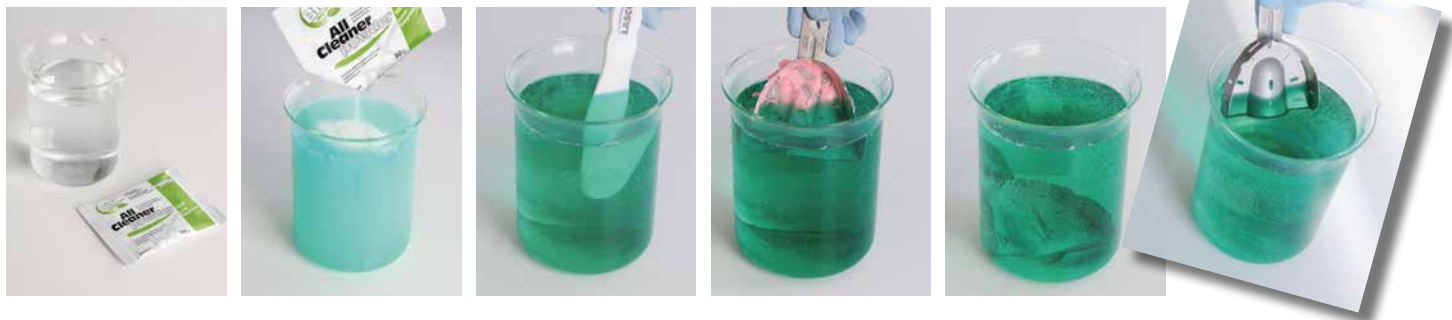


## LIQUID

Simply dilute 1 volume of **All Cleaner concentrated** with 9 volumes of water for a ready to use solution.

**1:9 CONCENTRATED**

With 1 package of 1 litre you get

**10 litres  
OF SOLUTION**

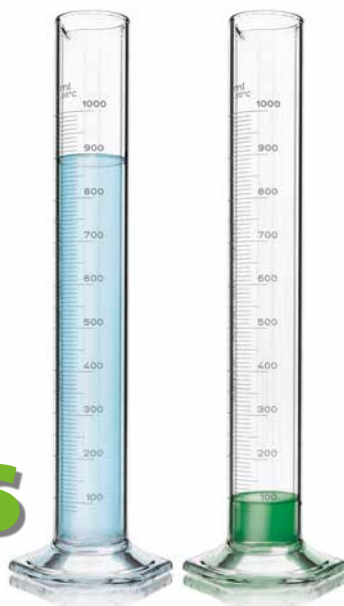

## POWDER

With the new single-use packaging **All Cleaner Powder** is easy and ready to use in seconds.

Simply pour the contents of one single-use bag (50g) into one litre of water to obtain a ready to use solution. The slight effervescence which develops helps the dissolution process.

With 1 package of 15 sachets you get

**15 litres  
OF SOLUTION**

# kind on surfaces

After submersions instruments and impression trays must be abundantly rinsed with water. **Instruments and impression trays must be subjected to appropriate processes of sterilization before their use.**

All Cleaner does not damage or dulls stainless steel surfaces.

**Taking into account aluminium trays' sensitivity to oxidation, we suggest short submersion timings.**



**COD**



# optimize time and space

## compatible with ultrasonic cleaners

All Cleaner solution can be used with ultrasonic equipments facilitating the removal of residuals and reducing further the submersion timings.

**If you dilute All Cleaner with warm water the required time to remove the residual are further reduced.**

## dental clinic



ALC100  
1 Lt.  
(=10 litres)

ALC500  
5 Lt.  
(=50 litres)

### VERSATILE

Dental Clinics and Laboratories will appreciate the great benefits offered by All Cleaner Powder in the storage spaces management.

You can find All Cleaner in Powder version (15x50g sachets enough for **15 litres** of solution) or concentrated liquid: 1 package of 1 liter to get **10 litres** of solution, or 5 litres package to get **50 litres** of solution.



ACX015  
15 x 50g.  
(=15 litres)

## dental laboratory

CLEGB4 - FEB. 2016

# ABSOLUTE HYGIENE

**Our instruments with perfectly smooth handle and our impression trays without point of soldering are easy to wash, clean and sterilize.**

## Zeffiro

The smooth handle (LASCOD PATENT) is the first step toward achieving efficient instruments cleaning.



## Exploit

Without welding and easy to clean, the first guarantee for avoiding cross-contaminations.

# LASCOD

For further information activate the QR Code reader on your smartphone.

