super etch

37% PHOSPHORIC ACID ETCHANT GEL

ÄTZGEL AUS 37%IGER PHOSPHORSÄURE ÁCIDO FOSFÓRICO EM GEL A 37% ACIDO FOSFORICO GEL AL 37% GEL DE MORDANÇAGE A L'ACIDE PHOSPORIQUE A 37% GEL MORDENZANTE CON ACIDO FOSFORICO AL 37% **ETS GEL 37% FOSFORZUUR 37% FOSFORSYRE ÆTSGEL 37% FOSFORSYRE ETSEGEL 37% FOSFORIHAPPO ETSAUSGEELI** ΑΔΡΟΠΟΙΗΤΙΚΟ ΖΕΛΕ 37% ΟΡΘΟΦΩΣΦΟΡΙΚΟΥ ΟΞΕΟΣ ŻEL DO WYTRAWIANIA SZKLIWA I ZĘBINY 37%-OS FOSZFORSAV GÉL SÖÖVITUSGEEL (37% FOSFORHAPE) 37% FOSFORNA KISLINA ZA JEDKANJE V GELU 37% FOSFORSKĀBES KODINĀŠANAS GĒLS 37% FOSFORO RŪGŠTIES IŠĖSDINIMO GELIS LEPTACÍ GEL S 37% KYSELINOU FOSFOREČNOU LEPTACÍ GÉL S 37% KYSELINOU FOSFOREČNOU リン酸エッチャント37% 超級酸蝕劑







37% promotes strong bonding

Super Etch 37% wt phosphoric acid etchant for enamel and dentin

Easily controlled Super Etch gel flows under direction, not under gravity

Indications

- Etching dentin and enamel
- Selective enamel etching

In conjunction with all

- Adhesives
- Composites
- Compomers
- Glass ionomers
- Sealants
- Cements

To clean

Porcelain

SDI's etchant range consists of: mL g 1.2 mL Super Etch gel syringe 1.5 g Super Etch gel syringe 2 mL 2.5 g Super Etch gel syringe 5 mL 6.25 g Super Etch gel syringe 9.6 mL 12 g Super Etch Low Viscosity (LV) gel syringe 1 mL 1.25 g **Etchant Gel bottle** 15 mL 18.75 g **Etchant Liquid bottle** 15 mL 18.75 g Super Etch Jumbo syringe 25 mL 31.25 g

Super erch erch

37% wt phosphoric acid

30 to 40% wt phosphoric acid optimally etches in 15 to 20 seconds ⁽¹⁾. Lower concentrations require longer contact time, unnecessarily delaying procedures.

Selective enamel etching

Super Etch's high viscosity and easy to control flow allows precise placement. It is perfect for selective enamel etching procedures.

Blue tint gel

High visibility for precise selective placement.

Thixotropic gel

Super Etch non-slump gel offers precise selective placement. Agitate the gel to lower its viscosity.

Super Etch

Super Etch creates micropores in the enamel surface and removes the smear layer to unblock dentinal tubules. It promotes adhesive resin penetration and resin tag formation. Super Etch is vital for the restoration's mechanical retention.

Dye free Etchant

A clear dye free 37% liquid etchant is also available for patients who suffer dye allergies. acid etch is

Super Etch syringes

Super Etch offers an easy to use, leak free syringe. Direct extrusion facilitates faster procedures, economical dispensing and easier access into difficult to reach areas.

Super Etch Low Viscosity (LV)

LV is specialized for sealants with a half gel / half liquid formulation. Super Etch LV's high flowability achieves greater acid surface contact for deeper etchant penetration into pits and fissures. The deeper acid penetration strengthens the mechanical bond and the retention success of the sealant. Ideal for SDI's Conseal f and all other sealants.

Super Etch syringe tips

Super Etch LV tips - 27 gauge pre-bent for minute pit and fissure crevasses. Super Etch tips - 23 gauge straight tips.

instructions:

for direct placement of all restorations

Regular viscosity Super Etch





Blow gently with dry, oil free air for 2 seconds to evaporate solvent. Leave surface glossy

7 Light cure for 10 seconds



8.1 Place Wave / Wave MV / Wave HV fluoride releasing flowable composite **or**



8.2 Place Aura (anterior/posterior) or Ice (anterior/ posterior) or Rok (posterior) composite



- 3 Light cure for 20 seconds in increments of 2mm
 - Polish and finish

instructions: for pit and fissure sealants

Low viscosity Super Etch LV 1 Clean and isolate tooth Etch tooth surface with Super Etch LV for at least 30 seconds, no more than 60 seconds Wash thoroughly with water Completely dry tooth. Note: a dry field is essential for the following steps Directly inject or brush apply 5 Conseal f pit and fissure sealant 6 Light cure Conseal f for 20 seconds CAUTION: Ensure the dispensing tip hub is firmly attached to the Super Etch, Wave or Conseal f syringe by twisting the hub securely onto the syringe

TOS ROAD JOANS



Super Etch bulk kit 10 x 2mL (2.5g) syringes 50 single use disposable tips Reorder 8100045

Super Etch kit 3 x 2mL (2.5g) syringes 25 single use disposable tips Reorder 8100040

Super Etch mini kit 4 x 1.2mL (1.5g) syringes 20 single use disposable tips Reorder 8100039

Super Etch 5mL syringe 1 x 5mL (6.25g) syringe 5 single use disposable 23 gauge tips Reorder 8100055

Super Etch 12g syringe 1 x 9.6mL (12g) syringe 25 single use disposable tips Reorder 8100052

Super Etch LV bulk kit

10 x 1mL (1.25g) Super Etch Low Viscosity syringes 40 single use disposable tips Reorder 8100044

Super Etch Jumbo 2x25mL syringes 2 x 25mL (62.5g) Super Etch syringes

50 single use disposable tips Reorder 8100059

Disposable tips for Super Etch syringes 25 single use disposable 23 gauge tips Reorder 8100050

Disposable tips for Super Etch syringes 200 single use disposable 23 gauge tips Reorder 8100060

Disposable tips for Super Etch Low Viscosity syringes 20 single use disposable 27 gauge tips Reorder 8100061

Etchant gel, blue 1 x 15mL (18.75g) Etchant gel bottle Reorder 8100020

Etchant liquid, clear 1 x 15mL (18.75g) Etchant liquid bottle Reorder 8100025 (1) Reality, Volume 13, 1999, 1-232





