

OH-REP + Importeur: DÜRR DENTAL SCHWEZ AG, Grabenackerstraße 27, 8156 Oberhasli, ZÜRICH | TEL: +41 448850634

LUNOS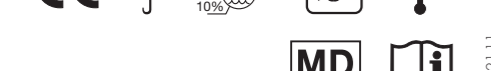

Lunos® Prophylaxepulver
Perio Combi
Lunos® Prophyl powder
Perio Combi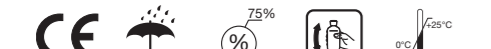

orochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Korwinstheim, DE
DÜRR DENTAL Company
Tel: +49 7142 705-0
www.duerdental.com
Made in Germany

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**EN Operating Instructions
Lunos® Prophyl Powder Perio Combi**
Prophy Powder Perio Combi (REF: CPZ640A1950)
Product description
“Prophylaxepulver Perio Combi” is a medical device. This product is only intended for professional dental applications and must be used exclusively by qualified persons in a clinical setting. This product is intended for professional use only. The „Prophylaxepulver Perio Combi“ is designed for sub- and supragingival application in commercial powder jet devices. Please note the operating instructions of the powder jet device before use of the prophylaxis powder. The formulation of “Prophylaxepulver Perio Combi” is free of animal ingredients, gluten, lactose, fructose and microplastics.

Indication
This product is recommended for:
• Removal of periodontal biofilm in the scope of periodontal maintenance therapy (recall therapy) with a suitable subgingival nozzle or the Perio Tip from Dürr Dental
• Cleaning implant surfaces to support treatment of mucositis or peri-implantitis after completed initial therapy
• Gentle removal of plaque at the sulcus and particularly sensitive dental surfaces
• Removal of slight discolorations in supragingival regions
• Removal of plaque on orthodontic appliances (brackets)
Prophylaxepulver Perio Combi is suitable for children and adults. This product does not have a cariogenic effect. Do not swallow.

Contraindication
Patients suffering from chronic airway diseases (e.g. bronchitis or asthma) must not be treated with a powder-water jet device since the powder present in the aerosol can lead to difficult breathing. Likewise, the product should not be used in patients with hypersensitized enamel and white spots. The treatment of deep periodontal pockets can lead to bacteraemia. Therefore, suitable measures and restrictions must be undertaken for the treatment of risk patients (endocarditis, pregnant and lactating women, infectious diseases, immunodeficiency (neutropenia, agranulocytosis, diabetes, haemophilia), patients on treatment (radiation therapy, chemotherapy, antibiotic therapy)).

Warning
Do not swallow. Avoid contact with the eyes. In case of contact with the eyes, rinse with plenty of water. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state. The product is intended for use on human subjects.

Precautions
Use appropriate protective equipment when working with “Prophylaxepulver Perio Combi”. The patient and operator need to wear suitable eye protection, e.g. protective goggles. The operator needs to wear suitable protective equipment in the form of mouth nose protection and suitable protective gloves. Use a suitable spray mist aspiration apparatus, e.g. the Dürr Dental Prophylaxepulver Perio Combi. The prophylaxis powder may only be used by appropriately trained dental specialists. Please make sure you do not direct the jet of the powder jet device directly at fixed dental restorations and fillings as this may damage or dislodge the spray. Do not directly spray the patient. The jet of the powder during treatment. Direct exposure of the soft tissue may damage the gingiva. Also there is a risk of emphysema arising in the tissue and/or of embolism from air being introduced into blood circulation. To protect sensitive skin, please apply Vaseline to the patient’s lips before commencing treatment. After completion of treatment, clean the patient’s face with a moist disposable cloth, e.g. the Lunos® Prophylaxepulver Wellness Cloth. Please advise your patient that the ingestion of food or beverages (tea, coffee or other hot drinks) within the two to three hours after completion of the treatment may cause renewed discoloration of the teeth. Clean the surfaces of the dental unit with a moist cloth following the use of the prophylaxis powder.

Preparation
Please remove all residual powder from the powder chamber of the powder jet device before refilling it with “Prophylaxepulver Perio Combi”. Shake the closed bottle before removing any “Prophylaxepulver Perio Combi”. Remove the tamper-proof seal after removing the powder. For this purpose, take the lid off the bottle, pull off the seal and then screw the lid back on the bottle. It is essential that you make sure that the powder chamber is completely dry before filling it. There must be no residual moisture in the chamber, e.g. from reprocessing measures. When filling the powder chamber with powder, you need to make sure not to exceed the maximum filling level of the powder chamber.
To minimise the amount of dust generated when you fill the chamber, it is recommended that you use the rocker and compress the bottle tightly.

Storage and shelf life
Please make sure that the bottles are carefully closed again immediately after removing the powder so that the prophylaxis powder does not absorb any moisture. Store in enclosed, well-ventilated rooms away from moisture and bright light between 0°C and 25°C with a relative humidity of up to 75%. Do not store in direct sunlight. When the storage conditions are observed, the shelf life is 36 months in the sealed container and 12 months after opening.

Disposal
This product is in accordance with the local official regulations.
Composition
Prophy Powder Perio Combi: Trehalose, excipients
Dürr Dental (Products) UK Ltd.
14 Linnet Way – Telford Way, Industrial Estate
Kettering, Northants NN16 8PS, United Kingdom
Tel: +44 1536526740

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