

PREMIUM WHITENING

CAVEX BITE & WHITE

AT-HOME
WHITENING
SYSTEMS

Safe, effective and
painless whitening
treatments under
your supervision

YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

CAVEX

CAVEX BITE&WHITE

More and more patients are looking for an effective, safe and painless way to whiten their teeth. Cavex developed various systems for at-home whitening, under the supervision of the dentist.

The whitening gel in the ABC Masterkit is based on 16% carbamide peroxide. Ready 2 Use contains 6% Hydrogen Peroxide Superior (HPS). The Impact Kit uses the active ingredient polyvinylpyrrolidone (PVP) peroxide.

Before and after treatment with the ABC Masterkit



Before: A4



After: B1

The Cavex Bite&White products are pH neutral. The systems are supplied and explained by the dentist and of course they fully comply with European laws and regulations. In short: at-home whitening systems for fantastic results, without risk and without sensitivity.

Technical insight:

During a whitening treatment, hydrogen peroxide (H_2O_2) breaks down into water (H_2O) and oxygen (O_2). The radicals released in this reaction break down the large colored molecules in the elements into small uncolored molecules. By repeating this several times, all colored molecules are broken down, the discoloration on the teeth has disappeared and the patient has radiant, natural white teeth.

PREMIUM WHITENING

ABC MASTERKIT

Is your patient looking for a thorough and long-term result with minimal sensitivity?

Then choose the ABC Masterkit: the most effective and complete whitening treatment using custom-made whitening trays. The ABC Masterkit is a 3-step system:



A

ACTIVATE with Cavex Bite&White StainLess (2 - 3 minutes)

This paste cleans the tooth surface thoroughly, so that the whitening gel comes into direct contact with the tooth enamel. In addition, StainLess increases the pH value in the mouth, making the whitening gel work faster.

B

BLEACH with Cavex Bite&White gel (30 - 60 minutes)

The whitening gel contains 16% carbamide peroxide which is equivalent to 6% hydrogen peroxide. The gel's ideal thixotropy ensures a perfect spreading over the tooth surface, while preventing the gel from running to the tooth neck and gums to prevent sensitivity. The added sodium fluoride strengthens the enamel and potassium nitrate minimizes any sensitivity.

C

CONDITION with Cavex Bite&White ExSense (5 - 10 minutes)

The unique composition of hydroxyapatite and a hydro-dispersing sol seals the tubules and microcracks in the enamel, minimizing sensitivity. In addition, ExSense contributes to the remineralization and restores the microhardness of the teeth.

ACTIVATE - BLEACH - CONDITION

STAINLESS

A The StainLess paste is used in the ABC Master kit prior to the whitening gel to make the whitening treatment even more effective. StainLess is also available separately.

The StainLess paste cleans the elements thoroughly, without adverse effect on tooth enamel or dentin. By cleaning the teeth thoroughly, the whitening gel comes into direct contact with the tooth enamel. In addition, StainLess temporarily increases the pH value in the mouth, making the hydrogen peroxide work faster. Colored molecules disintegrate quicker with a higher pH value, shortening the bleaching session.

Product features

- Thoroughly cleans the teeth from plaque and superficial stains
- Temporarily increases the pH value in the mouth for an even faster result

How to use?

Before bleaching, brush for 2-3 minutes with Cavex Bite&White StainLess and do not rinse.



PREMIUM WHITENING

WHITENING GEL

B This safe and effective whitening gel contains 16% carbamide peroxide (= 6% hydrogen peroxide)

The gel comes in syringes and can be easily applied in the whitening trays*. The whitening gel flows perfectly around the teeth, but does not flow to the necks of the teeth or gums because of the thixotropic properties. Therefore, the whitening result is evenly and the chance of sensitivity is minimal.

Product features

- 16% carbamide peroxide (= 6% hydrogen peroxide)
- Sodium fluoride strengthens the enamel
- Potassium nitrate minimizes any sensitivity
- Ideal thixotropic properties

How to use?

The gel is simply applied from the syringe into the whitening trays for the upper and lower teeth. Pour a drop the size of a pinhead into the tray per teeth. The patient then keeps the whitening trays in the mouth for 30 to 60 minutes.

*not included, to be produced separately.



A C T I V A T E - B L E A C H - C O N D I T I O N

EXSENSE

C This unique gel is applied after bleaching as the last step of the ABC Masterkit to prevent any sensitivity.

It provides fast and long-lasting relief from sensitive teeth. The unique composition of hydroxyapatite with a hydro-dispersing sol penetrates deep into the tubules and microcracks in the enamel, thereby completely closing sensitive areas.

Furthermore, ExSense contributes to the micro-hardness of the elements with the addition of 0.1% potassium nitrate. It offers long-term protection against sensitive teeth.

Product features

- Deep penetration of hydroxyapatite into the tubules and microcracks
- Prevents and minimalizes sensitivity of the teeth
- Accelerates the remineralization process
- Pleasant taste

How to use?

After the whitening treatment, rinse the whitening trays with water and apply Cavex Bite&White ExSense in the trays. Then keep the trays in the mouth for 5-10 minutes. Spit out any residue but do not rinse afterwards for a better effect.



PREMIUM WHITENING

READY 2 USE

Is your patient looking for a quick, effective and easy whitening treatment?

Then Ready 2 Use is the right choice. Ready 2 Use works super fast and can - as the name says - be used immediately. The universal whitening trays are pre-filled. The whitening gel is pH neutral and contains 6% Hydrogen Peroxide Superior (HPS).

HPS whitening gel works faster than any other hydrogen peroxide product. Because HPS is based on the most pure hydrogen peroxide it breaks down colored molecules more rapidly. The highly viscous gel becomes firmer during the treatment, keeping it in place. This keeps the gel in constant contact with the teeth. Ready 2 Use can be stored at room temperature because HPS is very stable.

How to use?

The universal whitening trays are pre-filled with whitening gel. The patient keeps the whitening tray in the mouth for 15 minutes. Repeat once a day for eight days in a row.

Product features

- 8 pre-filled universal trays for 8 days and 15 minutes bleaching per day
- 6% HPS for a faster and more effective whitening result
- Can be stored at room temperature
- Highly viscous gel for even distribution over and between the teeth
- pH neutral





IMPACT KIT

Is your patient looking for a user friendly do-it-yourself kit?

The Impact Kit offers this possibility. The kit consists of a double-sided tray that can be easily customized and a tube with enough whitening gel for about twenty whitening sessions. Together they form an easy and effective whitening treatment.

A special feature of this product is that the gel is temperature stable. Convenient, because that saves a lot of space in the refrigerator. The secret behind the stability is a polyvinylpyrrolidone (PVP) peroxide gel. Also, the thixotropic properties of PVP peroxide makes sure the gel stays perfectly in place for an evenly whitening result.

How to use?

The thermoplastic tray is immersed in hot water for one minute. The patient then places the whitening tray in the mouth and adjusts it by using the fingers to form the tray around the teeth. Then the patient applies a line of bleaching gel in the top and bottom of the tray and keeps it in the mouth for 30-60 minutes. The treatment can be repeated daily until the desired whitening result is achieved.

Product features

- Whitens upper and lower teeth simultaneously in just 30-60 minutes per day
- Sufficient whitening gel for up to 20 whitening sessions
- 6% PVP peroxide for stability
- Can be stored at room temperature
- pH neutral



PREMIUM WHITENING

CAVEX BITE & WHITE



-
Safe, effective and
painless whitening
treatments under your
supervision

YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

CAVEX

-
Safe, effective and painless whitening treatments under your supervision

Cavex Bite&White is a non-invasive cosmetic treatment for teeth whitening. Hydrogen peroxide is the basis of any effective whitening treatment. Cavex Bite&White products only has pure hydrogen peroxide that is stabilized to ensure the product's effectiveness for a long period of time

Order the Cavex Bite&White assortment through your depot.



Cavex Bite&White ABC Masterkit
(art.no BW090)

- 3 x 3 ml Cavex Bite&White gel
- 1 x 35 ml Cavex Bite&White StainLess
- 1 x 42 ml Cavex Bite&White ExSense
- whitening tray case
- shade guide
- instructions for use



Cavex Bite&White 3 Syringes (art. no BW007)

- 3 x 3 ml Cavex Bite&White gel and instructions for use

PREMIUM WHITENING



Cavex Bite&White Ready 2 Use
(art.no BW031)

- 8 x pre-filled whitening tray (3 ml), shadeguide and instructions for use



Cavex Bite&White Ready 2 Use Bulk
(art.no BW036)

- 50 x 3 ml pre-filled whitening tray and instructions for use



Cavex Bite&White StainLess (art.no BW070)

- 1 x tube of 35 ml



Cavex Bite&White Refill (art.no BW10)

- 1 x 3 ml Cavex Bite&White gel and instructions for use



Cavex Bite&White Bulk Refill (art.no BW15)

- 10 x 3 ml Cavex Bite&White gel and instructions for use



Cavex Bite&White Impact Kit
(art.no BW040)

- 1 x tube of 40 g gel, 1 x Boil&Bite whitening tray and instructions for use

YOUR
IMPRESSION
IS OUR
SPECIALTY

